



QUICK REFERENCE GUIDE

INTERPRETATION OF RESULTS


RESULTS

(see results pictures)

Negative: One pink/purple colored line appears in the rectangular window near the Control mark. No colored line near the Test mark means the TSH level is negative, and below the cut-off of 5 μ IU/mL.

Positive: In addition to the pink/purple line by the Control mark, a second pink/purple colored line seen near the Test mark means the TSH level is positive, and above the cut-off of 5 μ IU/mL

PLEASE NOTE: In addition to the pink/purple line by the Control mark **ANY PINK/PURPLE** line that is seen near the Test mark of the cassette within the 10-minute time is considered a positive result. The intensity of the line does not matter.

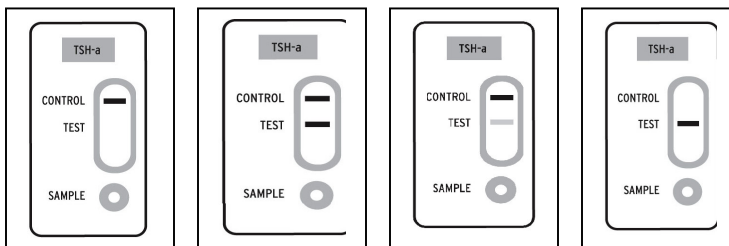

Negative

Positive

Positive

Invalid

Invalid: A pink/purple line should always appear near the Control mark. If there is no pink/purple line seen near the mark, the test is invalid. Do not report the result. In this case the test should be repeated with a new cassette or call 1-610 942-8971.

REPORTING RESULTS

The results of this test should be reported to a physician for individual interpretation and managing the symptoms.