



CLIA
WAIVED

- Studies estimate 13 million Americans are undiagnosed and needlessly suffer from the treatable hypothyroidism disease.
- 1 out of every 8 American women will develop a thyroid disorder.
- An estimated 20% of all chronic depression cases are associated with undiagnosed hypothyroidism.

YOUR RAPID THYROID STIMULATING HORMONE SCREEN

RESULTS IN 10 MINUTES

thyro
test[®]
Rapid Thyroid Stimulating
Hormone Screen



- ThyroTest is the fastest diagnostic device available to physicians to screen for Hypothyroidism.
- ThyroTest is an accurate diagnostic screening device that helps patients with common hypothyroid symptoms such as fatigue, weight gain, loss of libido and depression, gain vital diagnostic results in the best setting possible — **the physician's office.**
- Physicians and patients are no longer required to wait several days for their lab results.
- ThyroTest allows physicians to accurately detect abnormal/elevated levels ($> 5\mu\text{IU/mL}$) of TSH in 10 minutes.

Wampole Laboratories

 inverness medical
professional diagnostics group




Rapid Thyroid Stimulating Hormone Screen



ThyroTest arms physicians with accurate screening results in minutes and helps educate patients of their results in the same office visit.

PROCEDURE:

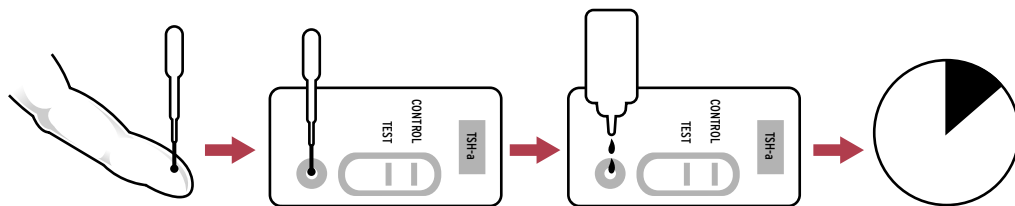

Rub the end of finger and pierce with lancet. Wipe away first drop of blood. Let blood fill to the line of the transfer pipette to collect fresh blood sample.


Hold the pipette vertically above the "S" well on the cassette and squeeze the pipette bulb to put the 2 drops of whole blood onto the cassette.

Immediately add six (6) drops of Test Buffer into the sample well (labeled "S" on the Test Cassette).

Read results in 10 minutes.

IMPORTANT:
Do not read after 12 minutes.

RESULTS

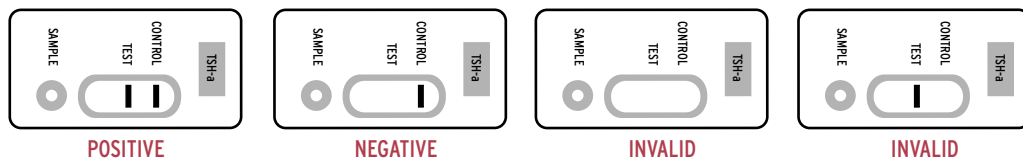

- The first and only CLIA Waived rapid TSH diagnostic screening test in the United States
- CLIA Waived
- Reimbursable

SPECIFICATIONS:

- Rapid Results: 10 Minutes
- Ease of Use: CLIA Waived
 - Easy to read test results
 - Simple 3 step process
- Accurate Results: 95.5% accurate (97.28% specificity, 81.25% sensitivity)
- Convenient: Whole Blood (finger stick)
 - 18 month shelf life
 - Room Temperature Storage



Wampole Laboratories
2 Research Way
Princeton, NJ 08540
800-257-9525 (T)
609-627-8013 (F)

Complete Packaging:

- Product Number 1020
- Test Kit Contains 20 - Tests w/ pipette
- 20 - Lancets
- 1 - Set of +/- controls
- 1 - 6 mL buffer solution

Wampole Laboratories



www.wampolelabs.com
www.thyrotest.com